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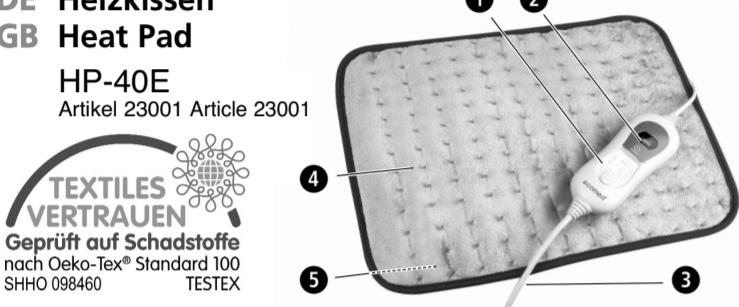
GB

Safety instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

Power supply

- Before connecting the heat pad to the power supply, please check that it is switched off and that the mains voltage of your power supply corresponds to the voltage listed on the label.
- Keep mains cable and control unit away from heat, humidity and liquids. Never touch the mains plug when your hands are wet or when you are standing in water.
- Only operate the heating pad with the corresponding switching unit **SS02**.

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- The device complies with the guidelines on electromagnetic compatibility. If you wear a pacemaker, this could be affected. Consult your doctor and the manufacturer of your pacemaker before using this device.
- If you have any medical concerns, please discuss using the heat pad with your doctor before using it.
- Do not use the device on children or persons who are disabled, asleep or insensitive to heat (persons who cannot react on overheating).
- Children below the age of 3 years are not allowed to use this device, because they are not able to react on overheating.
- The device must not be used by young children over the age of 3 years, unless the control unit has been adjusted accordingly by its parents or legal guardians or unless the child has been well instructed how to use the control unit properly.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!
- Do not use the heat pad on parts of the body which are swollen, inflamed or injured.
- Consult your doctor, if you have pain in muscles or joints for a prolonged period of time.

Operating the device

- Only use the heat pad as intended and in accordance with the instruction manual.
- The warranty will become void if it is used improperly.
- Do not use the unit for animals.
- This heat pad is intended only for private use and not for commercial purposes or for use in hospitals.
- The heating pad is subject to aging. Check heat pad, mains cable and control unit carefully for damage before each use. A faulty unit must not be used.
- Do not use the appliance if it is damaged, if its power cord is damaged, if it is not in perfect working order, or if it has been dropped or fallen into water. Send the unit to the service centre for repair to avoid potential hazards.
- Keep the unit out of rain and do not let it get wet. It must not be used out-of-doors.
- Never place the unit immediately next to a heater or radiator.
- Ensure that the unit does not come into contact with pointed or sharp objects and do not push pins into it.
- The heat pad must not be used if it has been folded, bent or crushed.
- Do not use the heat pad underneath quilts, blanket or pillows or cushions
- Do not use the heat pad when wet and only use it in dry environments (not in the bathroom or similar environments).
- The control unit must not be placed on or under the heat pad during operation and it must not be covered.
- Do not sit on the heat pad, but place it like a cushion on the body part, as required.
- Do not leave the heat pad unattended. Always switch off the heat pad when not in use and pull the plug out of the socket. Never use a time switch.

Operation

Insert the mains plug into the socket and switch on the heat pad. To do this move the switch on the control unit **①** to position **1**, **2** or **3**, as required. The switch clicks into each position. When the appliance is switched on, the regulating level appears on the operating control display **②**. The higher the regulating level, the higher the temperature.

To switch off the unit, move the switch completely back to position **0**. If the operating control display goes out, the device is switched off.

Move the selector switch to position **1** or **2** before falling asleep or any continuous use. The heat pad will switch off automatically after approx. 90 minutes of continuous use. Unplug the unit from the mains outlet once you have finished using it.

Switch positions

0 = the heat pad is switched off, the operating control display is off.

1 = middle heat level, the operating control display shows "1".

2 = middle heat level, the operating control display shows "2".

3 = highest heat level, the operating control display shows "3".

Automatic cutout

The heat pad is equipped with an automatic cutout.

The switch-off unit cuts off after 90 minutes of operation.

Return the switch to position **0**. However, to switch the heat pad on again, move the switch back to position **1** and then move it to the heat level required.

Cleaning and maintenance

- Before cleaning the pad, unplug it from the mains outlet and let it cool down for at least ten minutes.
- The heating pad is equipped with a detachable cable. Disconnect the plug **③** and remove the cable from the heating pad.
- You may wash the heating pad gently by hand. The best way to do this is to put the heating pad in a bath with lukewarm water and some mild detergent and then gently squeeze it out.
- To dry stretch the pad out on a flat surface.
- Store the heating pad several times to remove all the detergent.
- Store the unit in a clean and dry place, laid out flat with nothing covering or resting on top of it.
- Only use the appliance once it is completely dry.

Disposal

This product must not be disposed of together with domestic waste.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or communal collection point so that they can be disposed of in an environmentally acceptable manner.

Consult your local authority or your supplier for information about disposal.

Scope of supply

The following parts are included:

- 1 heat pad **HP-40E / HP-46E** including switching unit
- 1 instruction manual

If you notice any transport damage during unpacking, please contact your supplier without delay.

Technical specifications

Name and model

ECOMED Heat Pad HP-40E / HP-46E

Power Supply

: 220-240 V~ 50 Hz

Heat output

: 100 Watt

Switching levels

: 0 - 1 - 2 - 3

Automatic switch off

: after approx. 90 min

Operating conditions

: Only use in dry rooms as described in the instruction manual

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