DE Personenwaage PS-72E GB Personal Scale PS-72E

FR Pèse-personne PS-72E
IT Bilancia pesapersona PS-72E
ES Báscula personal PS-72E
PT Balança pessoal PS-72E
NL Personenweegschaal PS-72E
FI Henkilövaaka PS-72E
SE Personvåg PS-72E

GR Προσωπική ζυγαριά PS-72E
CZ Váhy koupelna PS-72E
HU PS-72E Fürdőszoba mérlegek
PL Wagi osobowe PS-72E
TR Banyo ölçekler PS-72E
RO Cântare personale PS-72E
RU Весы бытовые PS-72E

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Device and controls

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IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

1 Safety Information !

- Do not place the scale on an uneven surface. Never step onto a corner of the scales. When weighing yourself, do not stand on one side or the edge of the scales.
- Do not stand on the scale with wet feet. Do not stand on the scale wearing socks.
- The scale has been manufactured for domestic use. It is not suitable for commercial use in hospitals or other medical institutions.
- The scales have a measuring range up to 150 kg, 330 lb. Do not overload the scales.
- Handle the scale with care. The scale must not be subjected to impacts or vibration. Do not drop the scale. Keep the scale away from water.
- Do not attempt to repair the device yourself in the event of a malfunction since this will invalidate the warranty. Repairs should only be carried out by authorised service centres.
- If the scale is not going to be used for a long period, please remove the batteries.
- **WARNING!** Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation! Children shall not play with the appliance.
- Keep batteries out of children's reach! Do not disassemble batteries! Do not attempt to recharge these batteries! Do not short circuit! Do not throw into a fire!

2 Items supplied

- 1 Glass Personal Scale • 1 Type CR2032 3V lithium coin cell battery • 1 Instruction manual

3 Insert/replace batteries

The insulating strip must be pulled out of the battery compartment **1** on the underside of the unit before the scale can be operated. Replace the batterie if the battery change symbol "**LO**" appears in the display **3** or if nothing appears in the display after you have stepped on the device. Please note polarity (positive pole upwards; facing towards the lid).

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the battery before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop.

Cd Hg Pb

Batteries containing pollutant substances are marked as follows:
Cd = contains cadmium
Hg = contains mercury
Pb = contains lead

4 Change the unit of weight

You may have your weight displayed in kilograms (kg) or pounds (lb). Press button **2** until the unit you wish is shown.

5 Weighing "Step-on"-function

1. Make sure the scales are on a firm, level surface. If you only want to weigh yourself but have moved the scales just before, you must first initialise the scales. To do this, press your foot on the middle of the surface of the scales. "**0.0 kg**" appears in the display. **When the scales switch off, they are ready for the "step-on" function.**

This procedure is not necessary if you have not moved the scales.

2. Step onto the scales and stand still. The device switches on automatically and the display **3** briefly

shows "**0.0 kg**".

3. Your weight is measured, flashes twice and is then finally displayed.
4. Step down from the scales. The display changes to "**0.0 kg**". The scales switch off automatically after 10 seconds.

6 Error messages

ERR = The scale is overloaded. **LO** = The battery is low and must be replaced.

If your personal scales do not function to your satisfaction, check the following before contacting a service centre:

- Check whether the batteries have been inserted correctly.
- Check whether the correct unit of weight has been selected.
- Check whether the scales are freestanding on a solid and even surface. They must not be touching a wall or any other object.
- Weigh yourself again.

7 Care and maintenance

Never use aggressive cleaning agents or stiff brushes. Clean the scale using a soft and lightly moistened cloth. Never use abrasive cleaning agents or alcohol. Do not allow any water to get into the unit. Do not use the unit again until it is completely dry.

8 Technical specifications

Name and model	: ecomед Personal Scale PS-72E
Voltage supply	: 3 V \approx , 1 x 3 V lithium coin cell battery type CR2032
Measuring range	: up to 150 kg, 330 lbs
Graduation	: 100 g or 0.2 lb
Autom. switch-off	: after approx. 10 Sek.
Dimensions (LxWxH)	: approx. 30 x 30 x 2 cm
Weight	: approx. 1.5 kg
Item number	: 23510
EAN number	: 40 15588 23510 8

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

The current version of this instruction manual can be found under www.medisana.com

9 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **ecomед** products is two years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be corrected free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear (batteries etc.).
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

ecomед® is a trademark of  MEDISANA AG
Jagenbergstraße 19 • 41468 NEUSS • GERMANY
E-Mail: info@medisana.de • Internet: www.medisana.de

The service centre address is shown on the attached leaflet.

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ist eine Marke der  MEDISANA AG