DE Akku-Wärmecape

GB Battery Heat Cape

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PT Capa térmica a bateria

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IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation and operation.

LOT LOT number

Manufacturer

Protection category II

Do not use the heat cape when it is folded or creased

Do not puncture the device

Not suitable for children under 3 years

The heat cape can be cleaned in a gentle / delicate wash cycle at max. 30°C.

Do not bleach

Do not dry the device in a tumble drier

Do not iron the device

Do not dry clean

- The device is not suitable for medical use (e. g. in hospitals).
- Only use the device for the specific purpose described in the instruction manual. Any misuse will void the warranty.
- Do not fold or crease the heat cape during use.
- Do not use the heat cape on children or persons who are disabled, asleep or insensitive to heat (persons who cannot react on overheating).
- Neither pierce the device with, nor attach safety pins or other spiky or sharp objects to the cape.
- Never pull, twist or crease the cable.
- Keep the device away from humidity and liquids! Never use a damp cape.
- Check the device carefully for damage before each use. Do not use the device if you notice wear, damage or signs of improper use.
- If a fault occurs, do not attempt to repair the unit yourself. Repairs must only be carried out by an authorized specialist dealer or other suitably qualified personnel.
- When the heat cape is switched on:
 - never put any heavy or big objects on it (like e. g. suitcases)
 - never put any sources of heat on it (like e. g. heating pads).
- In case you do not use the heat cape daily, disconnect the power plug of the heat cape **⑤** from the battery pack.
- If you use the heat cape for an extended period (e.g. during falling asleep), level **1** (lowest heating level) should be adjusted.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Cleaning and user maintenance shall not be made by children without supervision.
- Children below the age of 3 years are not allowed to use this device, because they are not able to react on overheating.

- The device must not be used by young children over the age of 3 years, unless the control unit has been adjusted accordingly by its parents or legal guardians or unless the child has been well instructed how to use the control unit properly.
- Never use the device unsupervised.
- When storing the heat cape, allow it to cool down before folding.
- Do not crease the heat cape by placing items on top of it during storage.
- The rechargeable battery pack **⑥** needs to be fully charged prior to initial use.
- Before connecting the adaptor to your power supply, please ensure that the supply voltage stated on the rating label is compatible with your mains supply.
- When the charging process is finished, disconnect the battery pack **⑥** and the charging unit **①** from the power supply.
- The heat cape may only be operated with the battery pack control unit (WE-221b) and the charging unit (model no.: RSS1002-101084-W2E) supplied.
- If the power supply cable is damaged, it may only be replaced by **MEDISANA**, an authorised dealer or by qualified personnel in order to avoid hazards.
- The supplied charging unit (model no.: RSS1002-101084-W2E) should not be used for charging any other non-rechargeable or rechargeable batteries or battery packs.

2.1**Scope of delivery and packaging**

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the appliance and contact your supplier or your service centre.

The following parts are included:

- 1 MEDISANA heat cape with rechargeable batteries **HP 626**
- 1 Battery pack with control unit
- 1 Mains adaptor / charging unit
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your supplier without delay.

**WARNING**

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

2.2**Product features**

The **heat cape with rechargeable batteries HP 626** for indoor and outdoor use provides soothing heat. The device features 4 adjustable temperature settings, an overheating protection system and an automatic shut-off after 90 minutes of continuous use. The long-lasting heat is ensured by the powerful, rechargeable lithium-ion-battery pack. The cape is machine washable at 30°C (gentle / delicate wash cycle).

3.1 Charging the battery pack

The battery pack ⑥ needs to be charged for at least 3 hours prior to initial use or after not having been used for a long time. To charge the batteries, connect the cable of the charging unit ① with the battery pack and a mains outlet. During the charging process the charging status LED ⑧ flashes green. When the batteries have been fully charged, the charging status LED ⑧ lights up in green constantly. You now may disconnect the charging unit ① from the mains and from the battery pack ⑥.

If you want to check the state of charge of the battery pack, disconnect the power plug of the heat cape ⑤ from the battery pack and press the on-/off-button ⑦. The charging status LED ⑧ lights up in green constantly for approx. 3 seconds, if the state of charge is 25 % or better. If the state of charge is below 25 %, the charging status LED ⑧ flashes 3 times in a second. You then should recharge the battery pack ⑥.

IMPORTANT HINTS

- Only charge the battery pack in a dry environment at room temperature. Never place any objects onto the battery pack.

• BATTERY SAFETY INSTRUCTIONS:

- Do not disassemble or dismantle the battery / battery pack!

Danger of explosion!

- Keep away from fire and heat sources! Danger of explosion!

- Do not dispose used batteries in the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

3.2 Usage

1. Insert the battery pack ⑥ into the integrated battery pack pocket ④. Make sure the connection socket for the power plug of the heat cape ⑤ on the battery pack is placed upwards.
2. Connect the power plug of the heat cape ⑤ and the battery pack ⑥.
3. Adjust the desired temperature level (1 to 4) by pressing the on-/off-button ⑦ (multiple). Each press increases the temperature level by one step. If level 4 is already adjusted and you press the on-/off-button ⑦ again, the device will be switched off.
4. Place the heating cape on your shoulders and secure the cape by fastening the snap buttons ③.

- 3.3 The timer function** After 90 minutes of sustained usage, the heat cape will switch off automatically. If you want to continue to use the device, press the on-/off-button ⑦ to select the desired temperature level.

4 Miscellaneous

- 4.1 Cleaning and maintenance**
- Before each cleaning, disconnect the battery pack ⑥ from the mains resp. from the heat cape. Remove the battery pack from the cape.
 - Never use strong detergents or hard brushes.
 - Wipe the battery pack with a dry cloth carefully only.
 - The heat cape may be washed in the washing machine at 30°C (gentle / delicate wash cycle) in a laundry bag.
 - Minor stains can be removed with a cloth or damp sponge with a little liquid mild detergent.
 - The heat cape cannot be tumble dried or ironed. The cape should dry on the open air while switched off. Only use it again when it is completely dry.
 - Too frequently washing may lead to colour changes or to signs of wear and tear. Only clean the cape in the washing machine if it is absolutely necessary.
 - Store the device folded out and without kinks in a dry room at room temperature. The cape should cool down before storage.

4.2 Disposal

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

**4.3
Specifi-
cations**

Name and model	: MEDISANA Heat cape with rechargeable batteries HP 626
Power supply	
Heat cape	: 8.4V === 4W
Charging unit (RSS1002-101084-W2E)	: Input: 100-240V~ 50/60Hz 0.6A Output: 8.4V === 1,2A; 10.08W max.
Battery pack (WE-221b)	: Li-Ion 7.2V==; 2,500mAh; 18 Wh
Dimensions	: approx. 42 x 49 cm
Weight	: approx. 272 g (incl. battery pack)
Item number	: 61165
EAN number	: 40 15588 61165 0

**In accordance with our policy of continual product improvement,
we reserve the right for technical and design changes without
notice.**

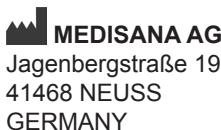
The current version of this instruction manual can be found under
www.medisana.com

**Warranty
and
repair
terms**


Please contact your supplier or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be corrected free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

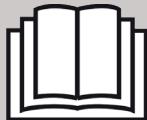The service centre address is shown on the attached leaflet.

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MEDISANA AG