- DE Körperanalysewaage
 GB Body analysis scale
 FR Pèse-personne d'analyse corporelle
 IT Bilancia con analisi corporea
 ES Báscula de análisis de la grasa corporal
 PT Balança com função de análise corporal
 NL Lichaamsanalyse weegschaal
 FI Henkilövaaka kehoanalyysitoiminnolla
 SE Personvåg med kroppsanalysfunktion
 GR Ζυγαριά προσώπων με λειτουργία ανάλυσης σώματος
 CZ Osobní váha s funkcemi rozboru těla
 HU Személymérleg testelemző funkciókkal
 PL Waga osobowa z funkcjami analizy stanu organizmu
 TR Beden analizi fonksiyonlu Baskül
 RO Cântar de persoane cu funcție körper analiza
 RU Весы бытовые с функцией определения параметров тела

Art. 23501

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IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

1 Safety Information !

- These scales are not suitable for people with pacemakers or other medical implants. The body fat information may be inaccurate in people with diabetes or other medical/physical limitations. The same also applies to very highly trained athletes.
- This device is not suitable for pregnant women!
- The scale has been manufactured for domestic use. It is not suitable for commercial use in hospitals or other medical institutions.
- Do not use the scale in a location where extremely high temperatures or humidity may occur. Keep the scale away from water.
- Do not expose the scales to direct sunlight!
- Do not place the scale on an uneven surface. Do not stand on the corners/edges of the scale. When weighing yourself, do not stand on one side or the edge of the scales.
- Do not stand on the scale with wet feet. Do not stand on the scale wearing socks.
- The scales have a measuring range up to 180 kg (396 lb / 28 st 4 lb). Do not overload the scales – this can cause irreversible damage.
- Handle the scale with care. The scale must not be subjected to impacts or vibration. Do not fall or drop anything on the scales.
- Do not attempt to repair the device yourself in the event of a malfunction since this will invalidate the warranty. Repairs should only be carried out by authorised service centres.
- Children shall not play with the appliance.
- Do not attempt to dismantle the scale, otherwise the warranty will be invalidated.

- **WARNING!** Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!
- Keep batteries out of children's reach! Do not disassemble batteries! Do not attempt to recharge batteries! Do not short circuit! Do not throw into a fire!

2 Items supplied

- 1 **ecomod** body analysis scale **BS-70E**
- 1 Type CR2032 3V lithium coin cell battery
- 1 Instruction manual

3 Insert/replace battery

The insulating strip may be pulled out of the battery compartment **6** on the underside of the unit before the scale can be operated. Replace the battery if the battery change symbol "Lo" appears in the display **1** or if nothing appears in the display after you have stepped on the device. Please note polarity (positive pole upwards; facing towards the lid).

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Please remove the battery before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop.

Batteries containing pollutant substances are marked as follows:

Cd = contains cadmium

Hg = contains mercury

Pb = contains lead

Cd Hg Pb

4 Change the unit of weight

You may have your weight displayed in kilograms (kg), pounds (lb) or stones (st). Press the unit key **2** until the unit you wish is shown.

5 Weighing

1. Make sure the scales are on a firm, level surface. If you only want to weigh yourself but have moved the scales just before, you must first initialise the scales. To do this, press your foot on the middle of the surface of the scales. "0.0 kg" appears in the display. When the scales switch off, they are ready for the "step-on" function. This procedure is not necessary if you have not moved the scales.
2. Step onto the scales and stand still.
3. Your weight is measured, flashes two times and is then finally displayed.
4. Step down from the scales. The scales switch off automatically after approx. 9 seconds.

6 Programming personal data

The scales can store data for up to 10 people: sex, age, height. The scales have preset values. These are: **kg, male, 25 years old, 170 cm.**

1. Press your foot on the middle of the surface of the scales.
2. Press the SET **4** button. The last used personal data is flashing in the display.
3. Now choose the desired unit of weight with the unit key **2**.
4. Press the SET **4** button. The memory location is flashing in the display. Now press **▲** - button **3** or **▼** - button **5** to select the desired memory location (P0 - P9) for the user profile.
5. Press the SET-button **4** to confirm the selection. Now the symbol for the sex is flashing in the display. **♂** = Man **♀** = Woman.
6. Set your sex by pressing the **▲** - button **3** or **▼** - button **5** to select the appropriate symbol in the display.
7. Press the SET-button **4** to confirm the selection. The preset height starts flashing in the display.
8. Select your height using the **▲** - button **3** and **▼** - button **5**.
9. Press the SET-button **4** to confirm the selection. The preset age starts flashing in the display.
10. Set your age by pressing the **▲** - button **3** or **▼** - button **5**.
11. Press the SET-button **4** to confirm the setting.
12. The settings are now concluded.

7 Weighing and measuring the body analysis values

1. Press your foot on the middle of the surface of the scales.
2. Press the SET **4**. The memory location flashes in the display. By pressing the **▲** - button **3** and **▼** - button **5**, you may choose the memory location (P0 - P9).
3. Your personal data is displayed.
4. Wait until **"0.0"** appears on the display. Step barefoot onto the scales and stand still. Place your feet on the electrodes **7**. First, your weight is displayed.
5. **"0000"** appears in the display and disappears from left to right. Now your weight flashes on the display. Afterwards, the measured values for bodyfat, body water, bone weight, muscle mass, BMR and BMI are displayed. The series of measured values is repeated twice in the display. The scales then switch off automatically. Step off the scales.

NOTE The data in the following table is for orientation purposes only. If you have any questions about your proportion of body fat, please consult your doctor.

Body fat proportion range in %

Age	female (%)				male (%)			
	low	normal	high	very high	low	normal	high	very high
10-39	14-20	21-25	26-31	32-38	11-17	18-23	24-29	30-36
40-55	15-21	22-26	27-32	33-38	12-18	19-24	25-30	31-37
56-80	16-22	23-27	28-33	34-38	13-19	20-25	26-31	32-38

Body water proportion in %

Age	female (%)			male (%)		
	low	normal	high	low	normal	high
10-15	< 57	57-67	> 67	< 58	58-72	> 72
16-30	< 47	47-57	> 57	< 53	53-67	> 67
31-60	< 42	42-52	> 52	< 47	47-61	> 61
61-80	< 37	37-47	> 47	< 42	42-56	> 56

8 Error messages

Err = The scale is overloaded. **Lo** = The battery is low and must be replaced.

Err2 = The bodyfat and body water values cannot be determined.

If your personal scales do not function to your satisfaction, check the following before contacting a service centre.

- Check whether the batteries have been inserted correctly.
- Check whether the correct unit of weight has been selected.
- Check whether the scales are freestanding on a solid and even surface.
They must not be touching a wall or any other object.
- Weigh yourself again.

9 Care and maintenance

Never use aggressive cleaning agents or stiff brushes. Clean the scale using a soft and lightly moistened cloth. Never use abrasive cleaning agents or alcohol. Do not allow any water to get into the unit. Do not use the unit again until it is completely dry.

10 Technical specifications

Name and model:	ecomed body analysis scale BS-70E
Voltage supply:	3 V  , 1 x 3 V Lithium coin cell battery / type CR2032
Display system:	Digital display
Memory:	for 10 persons
Measuring range:	up to 180 kg (396 lb / 28 st 4 lb)
Measuring deviation:	max. $\pm 1\%$
Graduation:	100 g or 0.2 lb or 1/4 st
Autom. switch-off:	after approx. 9 seconds
Dimensions (L x W x H):	approx. 30 x 30 x 1.75 cm
Weight:	approx. 1.6 kg
Item number:	23501
EAN number:	40 15588 23501 6

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

The current version of this instruction manual can be found under www.medisana.com

11 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **ecomed** products is two years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be corrected free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear (batteries etc.).
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

ecomed[®] is a trademark of

MEDISANA GmbH

Jagenbergstraße 19 • 41468 NEUSS
GERMANY

The service centre address is shown on the attached leaflet.

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