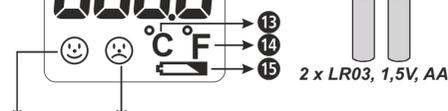
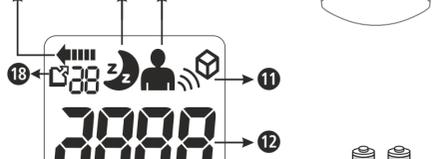
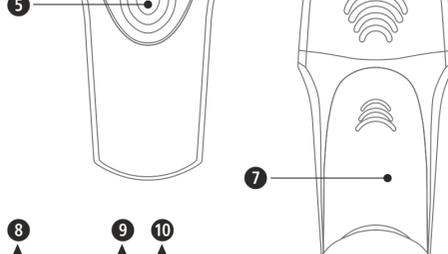
ecomed

23400 03/2019 Ver. 1.2

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GB Safety Instructions

Important Information! Retain for future use! Read the instruction manual carefully before using the device and follow the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

Intended use:
The **ecomed** infrared clinical thermometer **TM-65E** is suitable to measure the body temperature of humans. The measurement is carried out contact-free on the forehead. Additionally, the thermometer is suitable to measure the temperature of objects in the range of 0°C to 100°C.

- Only use the unit for the specific purpose described in the instruction manual. Any misuse will void the warranty.
- The unit is not designed for commercial use.
- Using this thermometer is no substitute for consulting a doctor. Consult your doctor in any case, before you start a medicamentous treatment.
- The thermometer and the patient must remain in a stable environment for at least 30 minutes before measuring the temperature. Do not take a measurement within 30 minutes after physical exercise or after having a bath.
- This device is not designed to be used by persons (including children) with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have inquired about the correct use of the device from a qualified person.
- Children must be supervised to ensure that they do not play with the device.
- The temperature in the forehead and temple area differs from the internal temperature, which is taken orally or rectally. Vasoconstriction, an effect which constricts the blood vessels and cools the skin, can occur during the early stages of a fever. In this case, the temperature measured by the **infrared clinical thermometer TM-65E** may be unusually low. If the measurement therefore does not match the patient's own perception or is unusually low, repeat the measurement every 15 minutes. As a reference, you can also measure the internal body temperature using a conventional oral or rectal thermometer.
- It is recommended to perform at least 3 measurements with at least 1 minute interval. If the measurement results differ significantly, accept the highest value as reference value.
- A control measurement using a conventional thermometer is recommended in the following cases:
1. If the reading is surprisingly low
2. For new-born infants up to 100 days old
3. For children under the age of 7 who have a weakened immune system or who react unusually in the presence or absence of fever
4. If you are using the thermometer for the first time or want to familiarise yourself with it
- Babies' skin reacts very quickly to the ambient temperature. Therefore, do not take their temperature with the **infrared clinical thermometer TM-65E** during or after breastfeeding, because the skin temperature may then be lower than the internal body temperature.
- There are no absolute body temperature standards. Keep record of your personal readings to allow your doctor to judge if a medical treatment is necessary or not.
- Do not change the construction of the device and do not attempt to repair the unit yourself if a fault occurs or the unit is damaged, as this will void the warranty. Contact your dealer and only have repairs carried out by authorized service partners.
- Do not use the thermometer if it is damaged.
- Do not open up the device. It contains small parts which might be swallowed by children.
- Do not use the thermometer in damp environments.
- Do not immerse the thermometer in water or other liquids. It is not watertight.
- The sensor should be cleaned after each use, see the instructions in **Cleaning and maintenance**.
- Do not let direct finger contact with the sensor.
- Do not shake or tap the thermometer. Do not drop it.
- Protect the device from direct sunlight, extremely high or low temperatures, dirt and dust.
- If the thermometer is not going to be used for a long period, remove the batteries.

Unit and controls
 Illuminated LCD display
 Signal LED (fever alarm)
 MODE-button
 SCAN-button
 MEM-button
 Battery compartment cover
 Symbol for running measurement
 Night mode

 Measurement of body temperature (Body mode)
 Measurement of object temperature (Object mode)
 °C Celsius
 °F Fahrenheit
 Battery exchange symbol
 Fever alarm symbol (>38°C/100.4°F)
 Symbol for normal temperature (<38°C/100.4°F)
 Memory display

NOTE
The night mode deactivates the key tones to avoid unnecessary noise.

Measuring the body temperature
1. Select a measurement mode (please refer to above chapter); Body mode or Body mode with night mode.
2. To start the measurement, point the sensor  in the middle of the patient's forehead at a distance between 0.5 and max. 3 cm.
3. If the patient's forehead has hair, sweat or dirt on it, remove this first so that the reading is as accurate as possible.
4. Press and hold the **SCAN-button**  and move the thermometer smoothly over the patient's eyebrow from the forehead to the temple to measure the maximum temperature.
5. Release the **SCAN-button** . You will hear a short beep.
6. The measurement result will be shown on the display .
After approx. 30 seconds without any further use the thermometer will switch off automatically to save battery power.

Items supplied and packaging
Please check first of all that the device is complete and is not damaged in any way. If in doubt, do not use it. Send it to a service point.
The following parts are included:
• 1 **ecomed** infrared clinical thermometer **TM-65E**
• 2 Batteries (AAA type, LR03) 1,5V
• 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING
Please ensure that polythene packing is kept away from the reach of children! Risk of suffocation!

Taking a temperature reading - quickly and accurately!
The **ecomed** infrared clinical thermometer allows precise, contact-free (approx. 0.5 to 3 cm distance) measurement of body temperature. The temperature is taken above the eyebrow between the forehead and the temple. The value of the energy radiated there is measured and converted to a temperature value. The reading is displayed within a few seconds. This enables you, for example, to take babies' temperature without disturbing their sleep.

Additional function
Apart from measuring body temperature, the **ecomed** infrared clinical thermometer can also be used to measure the temperature of objects (e.g. fluids, surfaces, ambient temperature). The temperature range for this function is between 0°C and 100°C (32°F - 212°F).

Getting started: Removing / Inserting batteries
For the use resp. if the battery exchange symbol  appears in the display after a certain period of use

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