MEDISANA®

home of wellness

- DE** Luft-Massagekissen ^{MPA}
- GB** Air massage cushion ^{MPA}
- FR** Coussin de massage à air ^{MPA}
- IT** Cuscino per massaggio ad aria ^{MPA}
- ES** Almohadilla de aire de masaje ^{MPA}
- PT** Almofada de massagem de ar ^{MPA}
- NL** Luchtmassagekussen ^{MPA}
- FI** Ilma-hierontatyyny ^{MPA}
- SE** Luft-massagekudde ^{MPA}
- GR** Μαξιλάρι αερομασάζ ^{MPA}

Art. 88933

Gebrauchsanweisung
Instruction Manual
Mode d'emploi
Istruzioni per l'uso
Instrucciones de manejo

Bitte sorgfältig lesen!
Please read carefully!
A lire attentivement s.v.p.!
Da leggere con attenzione!
¡Por favor lea con cuidado!

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IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

Protection category II

Lot number

Manufacturer

GB

1 Safety Instructions

MEDISANA®
home of wellness

Safety Instructions

about the power supply

- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Only insert the power supply unit into a socket when the appliance is switched off.
- Only use the power supply unit supplied for the massager.
- Keep the power supply unit, cable and appliance well away from heat, hot surfaces, moisture and liquids.
- Do not touch the power supply unit when you are standing in water and always take hold of the plug with dry hands.
- Do not take hold of a device if it falls into water. Pull out the power plug and the power supply unit from the socket immediately.
- Always switch off the appliance after use with the buttons on the control unit and unplug the power supply unit.
- To disconnect the unit from the power supply, always pull the plug out of the socket. Do not pull on the cable!
- Do not carry, pull or twist the device on the cable or the power supply unit.
- You must not continue to use the unit if the lead or the mains adaptor are damaged. For safety reasons, these parts may only be replaced by authorized service personnel. In order to prevent hazards, always send the unit in to the service centre.
- Ensure that the cable will not cause tripping. The cable must not be kinked, trapped or twisted.

specific groups of people

- This device is not designed to be used by persons (including children) with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- Children must be supervised to ensure that they do not play with the device.
- Do not use this unit to supplement or replace medical treatment. Chronic complaints and symptoms may be made worse.
- Before using the massager, consult your doctor regarding any unexplained pain, if you are undergoing medical treatment or using medical equipment.
- Do not use the **MPA** massager, or only after consulting your doctor, if:
 - you are pregnant,
 - you have a cardiac pacemaker, artificial joints or electronic implants,
 - you suffer from one or more of the following illnesses and complaints:
Circulation disorders, varicose veins, open wounds, bruises, broken skin, inflammation of the veins.
- If you experience any pain or discomfort during the massage, stop immediately and consult your doctor.

operating the unit

**The device is not to be used for commercial use or medical purposes.
If you have health concerns, consult your doctor before using the device.**

Only use the massager indoors.

Do not use the massager in damp rooms (e.g. when bathing or showering).

- The device must only be used for its intended purpose as described in the instruction manual.
- Using the mat for any other purpose invalidates the warranty.
- Do not use the appliance while you are asleep or when you are lying in bed.
- Do not use the appliance before going to sleep. The massage has a stimulating effect.
- Never use the appliance while driving a vehicle or operating a machine.
- Keep the device dry and never use it in wet or moist condition.
- Never put the unit down next to an electric stove or other sources of heat.
- Take care not to squash or jam the device.
- Do not allow the massage cushion to come into contact with pointed or sharp objects.
- Do not cover up the appliance when it is switched on. Never use it under blankets or cushions. There is a risk of fire, electric shock or injury.
- Never leave the unit unattended if it is connected to the power supply.
- The maximum operating time for one session is 60 minutes. Prolonged use reduces the service life of the appliance.

maintenance and cleaning

- The unit is maintenance free. If a fault does occur, just check whether the mains power adapter is properly connected.
- You may only clean the device yourself. If the device malfunctions please do not repair it yourself as this leads to the automatic expiry of all guarantee rights. Ask your specialist dealer and have repairs carried out by the service.
- Do not immerse the device in water or other fluids.
- If, nonetheless, liquid should penetrate the device, unplug the mains cable immediately.

Do not wash!

Do not dry clean!

GB

1 Safety instructions / 2 Useful Information

Thank you very much

Thank you for your confidence in us and congratulations!

You have gained a high-quality product from **MEDISANA** with the purchase of this **MPA** massager.

This device is intended for massaging the upper and lower back areas and the hip area.

To derive the most success and to enjoy your **MEDISANA MPA** massager for a very long time, we recommend that you read the following information on the use and care of the appliance carefully.

2.1 Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your supplier or your service centre.

The following parts are included:

- 1 **MEDISANA MPA** massager incl. control unit
- 1 Power supply
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.

If you notice any transport damage during unpacking, please contact your supplier without delay.

WARNING

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

3.1 How does the MPA massager work?

The **MPA** massager is intended for the targeted treatment of tense muscles in the upper and lower back areas and the hip area. The device uses innovative compressed air technology for this. The massaging frequency in the four air cushions can be set individually by selecting one of five compression times. A particularly intensive and relaxing effect is achieved with this special high-definition massage technology. The separate, optional vibration massage can be set to five levels of intensity. Compressed air massage and vibration massage can be used in combination or independently. A timer allows you to choose between four operating times.

You can thus use the **MPA** massager to create your own individual massage program.

3.2 Operation

Preparation

1. Place the massage cushion **9** on a high chair or seat. Make sure the position is correct. The cables are at the bottom. If necessary, use the velcro fasteners at the back to fasten it to a suitable back of a chair.
2. Insert the power supply unit plug **12** into the connection **11** on the massage cushion **9**.
3. Plug in the power supply unit **12**. Ensure that the power supply is easily accessible. The control unit **6** acknowledges this with a beep.
4. Seat yourself comfortably and familiarise yourself with the device functions.

Operating the unit

I/O button 1

1. Press the I/O button **1** to switch on the device. The device starts working straight away with compressed air massage and vibration massage. The lowest levels are set for the air massage frequency and vibration intensity. The preset operating time of 15 minutes appears on the display **5**. The LEDs above the three function buttons and the indication on the display light up in red.
2. Press the VIBRATION button **8** to switch off the vibration massage function. The LED goes out. The compressed air massage remains in operation. Press the AIR MESSAGE button **2** to switch off the air massage function. The LED goes out. The vibration massage remains in operation.
3. Press the I/O button **1** again to switch off the device. All LEDs and the display's illumination go out.

AIR MESSAGE button 2

1. Press the AIR MESSAGE button **2** to switch on the air massage function. The device starts working straight away with the lowest air massage frequency, i.e. with a compression time of 3 seconds. The massage effect is generated by building up pressure and releasing it again in the four air cushions **10** in the massage cushion.

2. You can use the **+/-** buttons **3** to change the air massage frequency setting, i.e. you increase or reduce the compression time. Five levels are possible (3 / 6 / 9 / 12 / 15 seconds). A changed setting is briefly indicated on the display.
3. Press the AIR MASSAGE button **2** again to switch off the air massage function. The red LED goes out.

VIBRATION button 8

1. Press the VIBRATION button **8** to switch on the vibration massage function. The device immediately operates at the lowest massage intensity, i.e. level 1. The massage effect is generated by four motors **10** in the massage cushion.
2. You can use the **+/-** buttons **7** to increase or reduce the massage intensity. Five levels are possible (1 / 2 / 3 / 4 / 5). A changed setting is briefly indicated on the display.
3. Press the VIBRATION button **8** again to switch off the vibration massage function. The red LED goes out.

TIMER button 4

Press the TIMER button **4** to select a specific operating time. You can choose between 15, 30, 45 and 60 minutes. A changed setting is indicated on the display. The remaining operating time is indicated on the display in minutes. Once the operating time has elapsed, the device switches off automatically.

**NOTE**

If you do not use the timer to set the time when operating the device, it will switch off automatically after 15 minutes.

**CAUTION**

Make sure that the maximum operating time of 60 minutes is not exceeded.

4.1 Cleaning and maintenance

- Before cleaning the appliance, make sure that the appliance is switched off and the power supply unit is disconnected. Allow the unit to cool down.
- Clean the massager with a slightly damp sponge only. Never use brushes, aggressive cleaning agents, petrol, thinners or alcohol. Rub the seat dry with a clean dry cloth.
- Never immerse the appliance in water to clean it and ensure water cannot get into the appliance.
- Wait until the device is completely dry before using it again.
- Undo any kinks in the cable.
- Take care when rolling up the mains lead to avoid damaging it.
- You should ideally store the device in its original packaging and keep it in a clean, dry place.

4.2 Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your supplier for information about disposal.

4.3 Technical specifications

Name and model	: MEDISANA Air massage cushion MPA
Voltage supply	: 100 - 240 V~ 50/60 Hz 500 mA
Power supply unit	: 12 V= 1200 mA
Power consumption	: approx. 14.4 W
Operating time	: 15, 30, 45, 60 minutes
Autom. switch off	: after approx. 15 minutes
Operating conditions	: only use in dry rooms
Storage conditions	: in a cool, dry place
Dimensions	: approx. 47 x 38 x 13.5 cm
Weight	: approx. 2.2 kg (not including power supply unit)
Power cable length	: approx. 1.8 m
Article No.	: 88933
EAN Code	: 40 15588 88933 2

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

GB 5 Warranty**MEDISANA®**
home of wellness**Warranty and
repair terms**

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

 **MEDISANA AG**

Jagenbergstraße 19
41468 NEUSS
GERMANY

E-Mail: info@medisana.de
Internet: www.medisana.de

The service centre address is shown on the attached leaflet.

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